

# Assembly- and operating instruction

## RÜCKENGURT 85



This back rest is delivered to attach at a standard rollator with tube diameter of 22mm or 25mm, as well as at Topro Troja, Handicare Gemino or Uniroll Topas Premium in a few steps.

Depending on the model choose the matching clamp out of the included bag and fix the back rest according to **Fig. 1 and 2** at the above grip tube of the rollator.

**Caution:** By mounting the fixing at rollators like Topro Troja, Handicare Gemino and Uniroll Topas Premium it could come to slightly but unavoidable damages of the surface by the screws. The damages are hidden by the clamps. Don't slide the mounted fixing on the tube (Danger of scratching marks)!

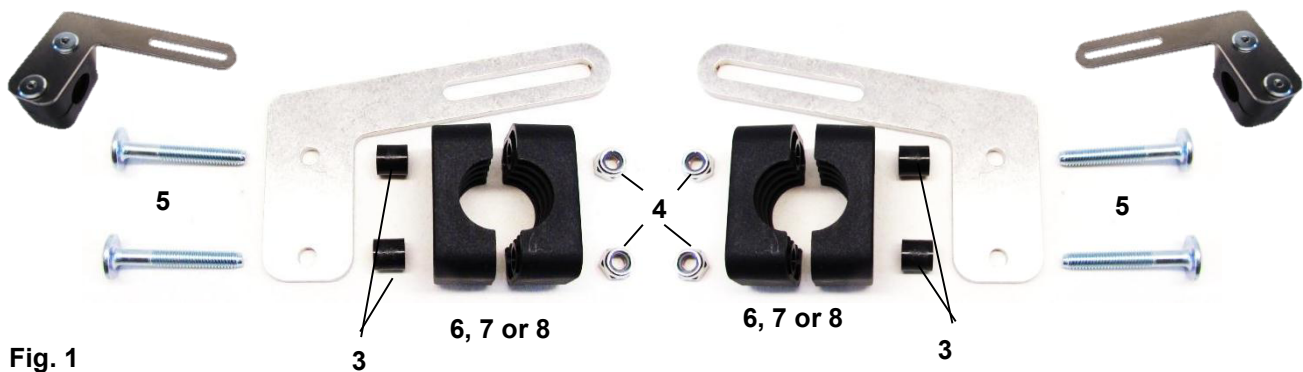


Fig. 1

At the end pull the belt (1) through the slots of the fixing-sheet (Fig. 2). Take care that the loop ribbon shows outwards. Now close the belt by help of the hook and loop at the length you need.



Fig. 2

**!** Don't use this item in a defect condition

This instruction is an integral part of each **® RÜCKENGURT 85** and must therefore be kept. If this item is passed on to a third party, this instruction has to be handed over as well. Potential returns or complaints can only be made in connection with this instruction and the receipt of purchase. Subject to modifications in the current