


Thank you for purchasing PROTEKTOR the umbrella for rollators and for your trust in our products. Before using this product the first time, carefully read the entire user manual. Check that all parts are included and follow all of the safety instructions.

Application / Purpose

PROTEKTOR, the umbrella for rollators offers perfect protection against rain and sun. The universal umbrella base ensures fast and easy installation to all standard 4-wheel rollators and wheelchairs. This allows you to carry your rollator or wheelchair safely with both hands. The side-mount leaves enough space for shopping bags, cane holders, back-belts, etc. Attention: Not suitable for Delta-walkers!

Delivery Content

1. Rollator umbrella with joint connectors
2. Umbrella base, pre-assembled with wedge clamps and 2 Allen screws M6x60
3. Mounting clamps for tube Ø 18/20/22/25 mm
4. Set of Allen screws M6x35/M6x45/M6x75
5. Allen wrench
6. User Manual

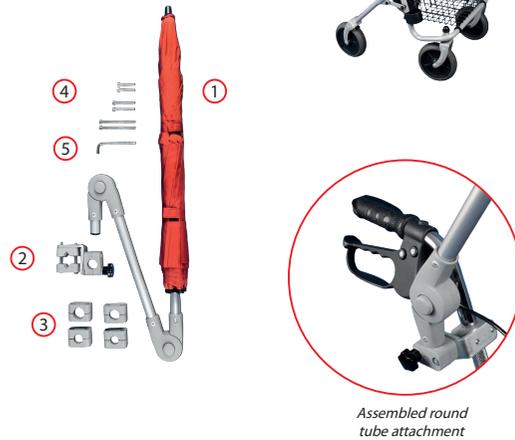

Technical Data

Weight: 1 kg
 Max. diameter (open): 105 cm
 Will fit for round tubes 18 - 25 mm DIA and oval tubes up to 60 mm

Safety Instructions






- Before using the umbrella the first time, make yourself familiar with the operation of the umbrella and the attachment. When using the PROTEKTOR, please take its height and size into consideration.
- Open the umbrella (with the automatic) never close to your body. To avoid injury, make sure, no other person is close around. Consider the space requirement of the open umbrella.
- Check before every use if umbrella base and umbrella are firmly attached.
- The umbrella base has to be attached such way to the rollator that the brake function is not impaired, regardless, if the umbrella is open or closed. Also, avoid any interference with lights, bell, cane holder, etc.
- Not suitable for Delta walkers (risk of tip-over!)
- Attach the umbrella-base in such a way to rollator that you can easily close and fold the umbrella. Shut the umbrella and tie the Velcro straps, before folding it together.
- Attention: Danger to pinch the fingers while closing umbrella or adjusting the joint-connectors!
- Always watch for an unobstructed view, in particular in road traffic. Consider height and size of the open umbrella.
- At strong wind, the umbrella should be closed.
- Do not move, nor carry the rollator by the umbrella base.



Assembly

The umbrella base is already prepared for oval tube attachment, at the right side* of the rollator (fig. a). For other tube shapes, replace clamps and Allen screws as needed.

- 1 Choose the matching clamps: oval tubes (base setup, fig.a), round tubes (18/20/22/25 mm, fig. b), or mix-match clamps for custom tubes (fig. d).
- 2 Choose the appropriate Allen screw (35/45/60/75 mm, fig. d) and attach the umbrella base via Allen wrench in desired position to the height adjustable vertical handle of the rollator.
- 3 Adjust the tilt angle of the umbrella base to the incline of the handle-tube, by first removing the Star screw completely. After adjustment, insert the Star screw back again into hole I or II of the bracket (fig. g+h).
- 4 To attach the umbrella to a horizontal tube, first remove Allen screw and Star screw. Thereafter attach the umbrella base again to the bracket via holes I and IV (fig. c+f).
- 5 Insert the umbrella with the shaft into the umbrella base and firmly tighten the Star screw. If required, re-tighten the Allen screw, to ensure a firm fit of the Umbrella with the base!
- 6 *Turn the umbrella base for installation on the left side of the rollator. Adjust the tilt angle of the umbrella base to the incline of the handle-tube, by first removing the Star screw completely. After adjustment insert the Star screw back again into hole III or IV of the bracket (fig. e).

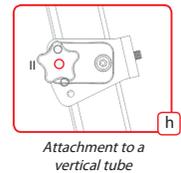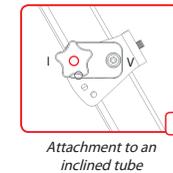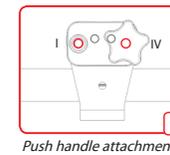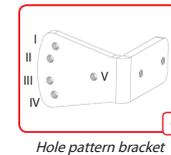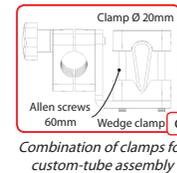

Operation Instructions

Bring the umbrella into the desired position, by pushing the spring-loaded button (3) on each of the joint connectors. In addition, the umbrella can be rotated the by loosening the Star screw of the umbrella mounting-clamp. Subsequently firmly tighten the Star screw again! Open the Velcro straps (1) of the umbrella and push the release button (2). Attention: the umbrella opens immediately. If not in use and to save space, the umbrella can be folded away on the joints.



All published information are based on nominal data and are subject to design and manufacture related tolerances.